

A.C. COSSOR & SON (SURGICAL) LTD
Accoson Works, Parkway,
Harlow Business Park, Harlow, Essex CM19 5QP

Telephone: +44 (0) 1279 433456
Facsimile: +44 (0) 1279 444018
E-mail: accoson@accoson.com
Websites: www.accoson.com
www.greenlight300.com



established 1859

CLEANING ACCOSON REUSABLE BLOOD PRESSURE CUFFS **and DEVICES**

Cuff (sleeve)

Remove other inflation system parts (bladder, bulb, valve) before washing. Remove any debris from the hook and loop fastener. Close the hook and loop fastener together. Do not wash cuff with natural fibres such as cotton or wool, as these fibres will cling to the hook and loop.

Machine wash in warm water (40 deg.C programme), then dry at warm setting.

Cuff may be hand washed with mild detergent and drip-dried.

Cuff can be gas sterilized. Do not expose cuff to steam or dry heat sterilization processes.

Bladder

Remove other inflation system parts (cuff, bulb, valve) before cleaning.

Apply a mild detergent to a slightly damp cloth and wipe clean. To avoid getting water inside the bladder, do not immerse tubes.

For bulbs, valves, and tubing, apply a mild detergent to a slightly damp cloth and wipe clean.

Devices

Both devices and the components of the cuff may be wiped clean using Clinell Universal Sanitising Wipes. These provide a very broad range of antimicrobial activity according to EN1276, prEN12054 and EN1500. Refer to www.clinell.com.